

Aicube -- Care with Love



Guangzhou SELAQUA Sanitary Ware Co., Ltd
Jintian Industrial Park, Huadong Town, Huadu District, Guangzhou City, Guangdong Province, China.
www.selaqua.com; www.aicubecare.com

Contact: Katie Peng
Sales Manager
M: +86 18620266308
E: katie@aicubecare.com

PRODUCT DETAILS

Type	Wall hung U-shaped shower seat with arms and backrest
Item Code	XSA-B420+B-BF340
Range	Bathroom Accessories
Materials	<ol style="list-style-type: none"> 1. The shower seat is constructed with super solid aluminum fixing bracket and super quality UF plastic. 2. The arms and backrest are made in high quality UF plastic and freely assembled in solid aluminum rack. 3. The end is capped with TPE plastic. 4. The fixing kits and fasteners are made of stainless steel.
Finish	Bright Chrome

Technical Details and Descriptions:

1. The shower seat and backrest armrests can be adjusted up and down. The backrest brings more comfortability. The armrest ensure stability when sitting on. Backrest and armrest are optional in separate.
2. The seat and armrests can be folded upward to save space.
3. The U-notch hole design in the middle of the seat is convenient for the elderly to clean their lower body while sitting.
4. The seat is with hydraulic damper kits making it soft open and anti-slip because of mat surface of seat
5. The seat can bear a load of 150 kgs and the armrests can bear a load of 40 kgs.

Standard and Warranty

Standard	ISO17966:2016
Warranty	<p>Expected service life of the shower seat is 10 years, provided that maintenance is carried out correctly.</p> <p>We makes a commitment to remedy material and manufacturing defects for a period of 3 years from the invoicing date</p>

Cleanning and Care Guide



1. The product is to be cleaned with hot water or a mild soapy cleaning agent.
2. We recommend that chalky deposits be removed from aluminum parts using a solution of 3:7 acetic acid: tap water. When the aluminum parts have been washed down with this solution, the parts must be wiped over with a damp cloth, wrung out in clear water. The product must be wiped down with a soft rag wrung out in clear water immediately after cleaning, as some materials may otherwise be discolored and stained, and the surface coating may deteriorate.

Warm Note: Do not use chlorine based, corrosive or abrasive cleaners as it may result in the surface being damaged.

IMAGE



TECHNICAL DRAWINGS

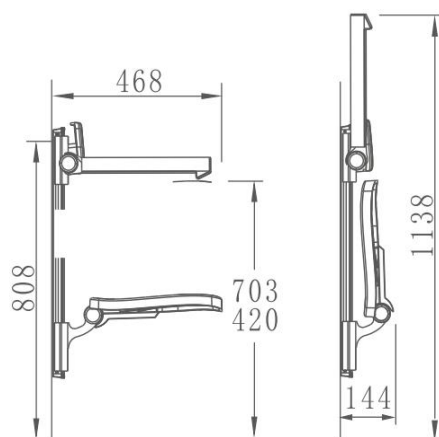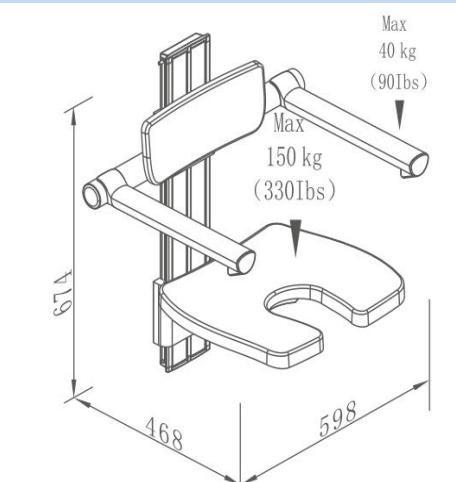

Full set with 2 carton box:

1. Shower seat in one box: 70x46x17cm 8.5kgs
2. arms with backrest in one box, 63x50x13cm. 6.5kgs;